
Sterilization Solutions

Euro Clinical Service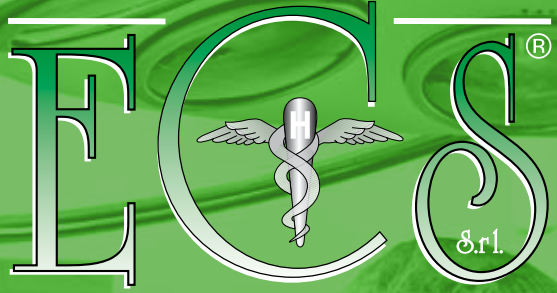

BIOLOGICAL TESTS

BIOLOGICAL TESTS

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Our biological systems BIO-CHECK are conceived to control the efficacy of the sterilization cycle performed.

The different versions of the biological indicators BIO-CHECK are recommended for the different kinds of sterilization process: [VH2O2], [STEAM], [EO], [FORM], [DRY]

BIO-CHECK Biological indicators contain bacterial spores on a paper support packaged inside a small plastic tube containing a glass ampoule filled up with a specially-formulated culture medium containing a coloured indicator, especially developed for the bacterial growth.

In the cap of the plastic tube of the BIO-CHECK biological indicators there is a filter that allows the sterilizing agent to enter inside the indicator itself. BIO-CHECK are designed to be used also in the test packets BIO-PACK. Each package of BIO-CHECK biological indicators has inside its referring certificate.

[REF] BIO-CHECK 98.B6 biological indicator suitable for the [VH2O2] sterilization - used spores Geobacillus stearothermophilus. The final reading of the result of the biological indicator BIO-CHECK 98.B6 is made after 48 hours of incubation.

NEW

[REF] BIO-CHECK 98.B5 rapid readout biological indicator has been created for the rapid and reliable monitoring of the sterilization process and it is suitable for the [STEAM] sterilization - used spores Geobacillus stearothermophilus. The final reading of the result of the rapid biological indicator BIO-CHECK 98.B5 is made after 3 hours of incubation.

NEW

[REF] BIO-CHECK 98.B4 biological indicator suitable for the [STEAM] sterilization - used spores Geobacillus stearothermophilus. The final reading of the result of the biological indicator BIO-CHECK 98.B4 is made after 24 hours of incubation.

[REF] BIO-CHECK 98.B2 biological indicator suitable for the [EO] sterilization - used spores Bacillus atrophaeus. The final reading of the result of the biological indicator BIO-CHECK 98.B2 is made after 48 hours of incubation.

[REF] BIO-CHECK 98.B7 biological indicator suitable for the [FORM] sterilization - used spores Geobacillus stearothermophilus. The final reading of the result of the biological indicator BIO-CHECK 98.B7 is made after 48 hours of incubation.

[REF] BIO-CHECK 98.B8 biological indicator suitable for the [DRY] sterilization.

NEW

BIOPACK RAPID and BIOPACK biological disposable packets for [STEAM] autoclaves with rapid and standard readout of the result.

INCUBATORS FOR THE COMPLETE RANGE OF RAPID AND STANDARD READOUT BIOLOGICAL INDICATORS.